



VIBRATING STEP SENSOR INSTRUCTIONS FOR USE

EASY TO USE.

- 1 Charge Step Sensor fully before first use.
Charging time will vary depending on temperature of the surrounding environment and situation.
Yellow indicator light will illuminate while band is charging to indicate unit is in charge mode. Yellow indicator light will turn off when unit is fully charged.
- 2 Remove adhesive backing from sensor and secure to insole of desired athletic or dress shoe.
- 3 Slip shoe on foot as normal.
- 4 Use the hook and loop strap to wrap the vibrating band around the leg just under the knee. The band should be snug, yet comfortable.
- 5 Nothing to turn on, simply check the connector between the two components to ensure a secure attachment. Enjoy your day.
- 6 At the end of the day, remove the vibrating band from leg and detach USB cable from sensor at connection port, keeping cable attached to band. Plug in to charge band using included Intellinetix charging block. The sensor itself can remain in the shoe for your next use.
- 7 Step Sensor can be worn all day, everyday
 - working
 - exercising
 - strolling
 - shopping

EASY TO SERVICE.

Brownmed warrants this product to be free from manufacturing defects. If your Step Sensor stops working within one year from date of purchase, we will replace it.

EASY TO CARE FOR.

- Spot clean as needed.
- Lay flat and allow to air dry.
- Keep charger opening dry while cleaning.
- For adult use only.
- Do not dry clean.
- Do not allow the product to get wet.



Available by prescription only

Need a replacement or extra parts?

Customer service is happy to help you.
Please call us at **1-877-853-5518**.

Show us how **you** vibe.
#Intellinetix

Learn more at intellinetixvibration.com



Executive:
101 Federal Street, 29th Floor
Boston, MA 02110
P 857-317-3354
F 857-317-3355

Sales:
4435 Main Street, Suite 820
Kansas City, MO 64111
TF 877-853-5518
P 816-581-7001
F 816-581-7005

Manufacturing:
1300 Lundberg Drive West
Spirit Lake, IA 51360
P 712-336-4395
F 712-336-2874

RM07504 Rev. 2

2019-0025