


UNIVERSAL THERAPY WRAP
+ HEATED VIBRATION HUB
INSTRUCTIONS FOR USE

EASY TO USE.

- 1 Lock Hub securely into Wrap.

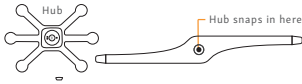

- 2 To turn on, simply hold button on the center of Hub for 3-5 seconds until the white light is illuminated.



- 3 Press right (single) button to control heat, red light illuminates. Press left (double) button to control vibration, orange light illuminates.

Controls heat



Controls vibration



- 4 Place Intellinetix Heated Hub + Wrap on area of concern, ensuring nodes of Hub surround targeted area of pain or muscle tension. Secure in place using hook and loop closures on Therapy Wrap.

- 5 Toggle between 3 levels of heat, vibration or combination of both to best address your area of concern.



3 levels of heat.



3 levels of vibration



3 levels of heat + vibration.

- 6 When finished, simply hold button down for 3-5 seconds to turn off.

EASY TO MAINTAIN.

Recharges in minutes with USB charger. Simply plug one end into AC wall adapter or powered USB port on alternative device and other end into the Hub. Lights will illuminate as unit charges. Orange/red light indicates full charge. Fully charged, Hub will operate up to 45 minutes.

EASY TO SERVICE.

Brownmed warrants this product to be free from manufacturing defects. If your Intellinetix Heated Hub + Therapy Wrap stops working within one year from date of purchase, we will replace them.

EASY TO CARE FOR.

- Spot clean as needed.
- Lay flat and allow to air dry.
- Keep charger opening dry while cleaning.
- For adult use only.
- Do not dry clean.

Need a replacement or extra parts?

Customer service is happy to help you.

Please call us at **1-877-853-5518**.

Show us how **you** vibe.
#Intellinetix

Learn more at intellinetixvibration.com



Executive:
101 Federal Street, 29th Floor
Boston, MA 02110
P 857-317-3354
F 857-317-3355

Manufacturing:
1300 Lundberg Drive West
Spirit Lake, IA 51360
P 712-336-4395
F 712-336-2874



WARNING: Lithium-ion Batteries and/or products that contain Lithium-ion Batteries can expose you to chemicals including cobalt, lithium nickel oxide and nickel, which are known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov.