(Revised: 6/10/25)

ARABIC

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Materiaal: synthetisch rubber

© 2025 Brownmed-SEAL-TIGHT, the SEAL-TIGHT logo, "The easy durable way to stay dry" and "Stay dry while getting wet" are trademarks or registered trademarks of Brownmed (877) 853-5518.
Rev. 1

ENGLISH

Instructions for Use

After removing your cast protector from the box, unfold completely.

Pull SEAL-TIGHT® Sport over cast by stretching the opening of the cast protector over your cast while pulling up. You will know it's at the right position on your arm or leg when the seal opening sits comfortably above the top cast edge.

Important note: The seal should be tight, after all, it's keeping water off of your cast. Never trim with scissors or puncture. It's ok if the bag appears oversized, as this allows room for large casts and easy movement.

Being careful not to submerge your leg past the seal, insert protector into the water while simultaneously holding a portion of the seal away from your skin with one finger. This will allow any excess air to escape easily from the bag. When the seal is approximately one inch from the water's edge, remove your finger so the seal fits snugly against your skin.

To Remove: Dry the protector thoroughly before stretching the seal over cast and remove. DO NOT remove by pulling on the bag.

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Spirit Lake, Iowa 51360 U.S.A.